

Step by step guide to using **podopro** UK™ LTD silicone molding putty



1. Measure equal quantities of Part PP20-A & Part PP20-B.



2. Mix both parts together in the hands for 1-2 minutes.




3. After 1-2 minutes an even colour is achieved.



4. Form the mixture into the appropriately shaped device, and leave in place to set for up to 4 minutes.



5. Ensure the orthotic is completely set before attempting to trim.

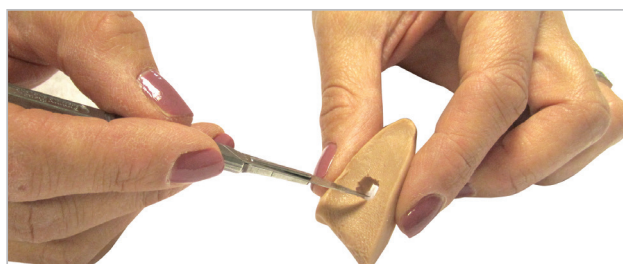

6. Can be trimmed if necessary with a scalpel after setting.



7. Final smoothing can be completed using a drill.



8. The material is very strong & resists tearing, allowing the user to produce more complicated, longer lasting orthotics.